

T3 FREEDOM MASK User Instructions

T3-TF-MSK



T3Gear.com

Misuse may result in sickness or death.

ESSENTIAL

Before use, wearer must read and understand these *User Instructions*. Keep *User Instructions* for reference.

The product has not been FDA or NIOSH cleared or approved.

The product has been authorized by FDA under an EUA for use as source control by the general public as well as by health care professionals in healthcare settings as to help prevent the spread of infection or illness during the COVID-19 pandemic.

This product is authorized only for the duration of the declaration that circumstances exist justifying the authorization of the emergency use of medical devices, including alternative products used as medical devices, during the COVID-19 outbreak, under section 564(b)(1) of the Act, 21 U.S.C. Â§ 360bbb-3(b)(1) unless the authorization is terminated or revoked sooner.

The product does not claim to be intended for use as a surgical mask or to provide liquid barrier protection.

The product is recommended against use in a clinical setting where the infection risk level through inhalation exposure is high.

The product is not stated or implied that the product is intended for antimicrobial or antiviral protection or related uses or is for use such as infection prevention or reduction.

The product is not intended to be a respiratory protective device, and therefore should not be used for particulate filtration.

The product is not labeled for use in high risk aerosol generating procedures.

BODY CONTACTING MATERIALS

The mask is made from elastic and cotton.

CLEANING INSTRUCTIONS

The mask may be cleaned in a washing machine on gentle setting and dried in dryer at the low setting.

BIOLOGICAL PARTICLES

This mask cannot eliminate the risk of contracting infection, illness or disease. OSHA and other government agencies have not established safe exposure limits for these contaminants.

USER CAUTION

Any risks associated with the use of the T3 Freedom Mask fall upon the user.

USE INSTRUCTIONS

1. Failure to follow all instructions and limitations on the use of this mask and/or failure to wear this mask during all times of exposure can reduce mask effectiveness and may **result in sickness or death**.
2. The particles which can be dangerous to your health include those so small that you cannot see them.
3. Leave the contaminated area immediately if dizziness, irritation, or

other distress occurs.

4. Store the mask away from contaminated areas when not in use.
5. Inspect mask before each use to ensure that it is in good operating condition. Examine all the mask parts for signs of damage. The mask should be disposed of immediately upon observation of damaged or missing parts. Immediately replace mask if damaged.

Dispose of used product in accordance with applicable regulations.

USE LIMITATIONS

1. This mask does not supply oxygen.
2. Do not alter, abuse, wash or misuse this mask.
3. Do not use with beards or other facial hair or other conditions that prevent a good seal between the face and the sealing surface of the mask.
4. Masks will not prevent contaminant entry through other routes such as the skin, which would require approved personal protective equipment (PPE).
5. This mask is designed for use by adults who are properly trained in their use and limitations. This mask is not designed to be used by children.
6. Individuals with a compromised respiratory system, such as asthma or emphysema, should consult a physician and complete a medical evaluation prior to use.

STORAGE CONDITIONS AND SHELF LIFE

Before use, store masks in the original packaging, away from contaminated areas, dust, sunlight, extreme temperatures, excessive moisture and damaging chemicals.

Storage temperature: -4 degrees to 86 degrees Fahrenheit

Maximum relative humidity: <80%

If mask becomes damaged, soiled or breathing becomes difficult, replace the mask.

FITMENT

Must be followed each time mask is worn.

1. Clean hands via washing with soap and water or application of hand sanitizer.
2. Remove mask from packaging and hold with straps facing upward.
3. Place mask straps around ears.

REMOVAL

Must be followed each time mask is worn.

1. Cup mask in hand to maintain position on face.
2. Still holding the mask in position, remove straps from ears.
3. Store mask properly.
4. Clean hands via washing with soap and water or application of hand sanitizer.